## Erkodur-al/ Erkodur-al 240

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### EN: Instructions for use

Erkodur-al (Ø 120 x 125 mm) and Erkodur-al 240 (Ø 240 mm), colorless, clear

The general purpose of Erkodur-al is the manufacture of appliances for dental and orthodontic treatment.

| Application (material)          | Intended use                     | Indication       | Patient groups    |
|---------------------------------|----------------------------------|------------------|-------------------|
| Aligner (Erkodur-al/-240) - MD  | Positioning of teeth             | Misaligned teeth | Adults, teenagers |
| Retainer (Erkodur-al/-240) - MD | Stabilization of tooth positions | Misaligned teeth | Adults, teenagers |

**Contraindications:** If an allergic reaction to plastics is already known, a specific allergy to Erkodur-al (Copolyester) should be ruled out.

**Caution:** Sensitive to moisture, close the aluminum bag tightly again immediately. Observe storage symbols.

**User:** Dental professionals

**Intended use:** Erkodur-al is recommended as a medical device exclusively for the above-mentioned purposes.

**Note:** The use is subject to responsibility and is based on the recommendation/prescription of the dentist. Allergic reactions are unlikely, but cannot be ruled out.

Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small fractures.

**Combination products:** Erkodur-al can be processed in all devices of the Erkoform and Erkopress series from Erkodent.

**Cleaning:** Dental splints can be cleaned with Oxydens cleansing tablets. Also with curd soap, water and a toothbrush. Toothpaste is not suitable. Do not heat above 50 °C.

Please rinse the splint with clear water before inserting it. As the user, please inform the patient about the cleaning options.

**Further information:** Thermoforming technique brochure and videos, [www.erkodent.com](http://www.erkodent.com)

**Serious incidents** in connection with Erkodur-al must be reported. The user must inform the competent authority and the manufacturer.

**Performance characteristics** (Circa values): Density 1.19 g/cm<sup>3</sup>, modulus of elasticity 1462 MPa, elongation at break 179 %, water absorption 0.5 %, temp. resistance 85 °C

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