Erkoflex-95

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Erkoflex-95 (Ø 120 + 125 mm und quadratisch 125 x 125 mm), transparent

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EN: Instructions for use

Erkoflex-95 (Ø 120 + 125 mm and square 125 x 125 mm), transparent

The general purpose of Erkoflex-95 is the manufacture of appliances for dental and orthodontic treatment.

Application (material)	Intended use	Indication	Patient groups
Occlusal splints - MD	CMD treatment	CMD	Adults, teenagers, children
Protective splints - MD	Protection after local surgery	Local surgery (e.g. implantation)	Adults, teenagers

Contraindications: If an allergic reaction to plastics is already known, a specific allergy to Erkoflex-95 (EVA) should be ruled out.

Caution: Observe storage symbols.

User: Dental professionals

Intended use: Erkoflex-95 is recommended as a medical device exclusively for the above-mentioned purposes.

Note: The use is subject to responsibility and is based on the recommendation/prescription of the dentist. Allergic reactions are unlikely, but cannot be ruled out.

Combination products: Erkoflex-95 can be processed in all devices of the Erkoform and Erkopress series from Erkodent. Erkoflex-95 can be combined with Erkoflex Sticks-95.

Cleaning: Dental splints can be cleaned with Oxydens cleansing tablets. Also with curd soap, water and a toothbrush. Toothpaste is not suitable. Do not heat above 50 °C.

Please rinse the splint with clear water before inserting it. As the user, please inform the patient about the cleaning options.

Further information: Thermoforming technique brochure and videos, www.erkodent.com

Serious incidents in connection with Erkoflex-95 must be reported. The user must inform the competent authority and the manufacturer.

Performance characteristics (Circa values): Density 0.93 g/cm³, modulus of elasticity 70 MPa, elongation at break 450 %, water absorption 0.2 %, temp. resistance 62 °C

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