Erkoflex-bleach/240

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The general purpose of Erkoflex-bleach is the manufacture of appliances for dental and orthodontic treatment.

Application (material)	Intended use	Indication	Patient groups
Fluoridation splints (Erkoflex-bleach/240) - MD	Caries prevention	Caries prophylaxis	Adults, teenagers, children
Bleaching splints (Erkoflex-bleach/240) - cosmetic application	Whitened teeth	Colored teeth	Adults, teenagers

Contraindications: If an allergic reaction to plastics is already known, a specific allergy to Erkoflex-bleach (EVA) should be ruled out.

Caution: Observe storage symbols.

User: Dental professionals

Intended use: Erkoflex-bleach is recommended exclusively for the above-mentioned purposes.

Note: The use is subject to responsibility and is based on the recommendation/prescription of the dentist. Allergic reactions are unlikely, but cannot be ruled out.

Combination products: Erkoflex-bleach can be processed in all devices of the Erkoform and Erkopress series from Erkodent.

Cleaning: Dental splints can be cleaned with Oxydens cleansing tablets. Also with curd soap, water and a toothbrush. Toothpaste is not suitable. Do not heat above 50 °C. Please rinse the splint with clear water before inserting it. As the user, please inform the patient about the cleaning options.

Further information: Thermoforming technique brochure and videos, www.erkodent.com

Serious incidents in connection with Erkoflex-bleach must be reported. The user must inform the competent authority and the manufacturer.

Performance characteristics (Circa values): Density 0.93 g/cm³, modulus of elasticity 70 MPa, elongation at break 450 %, water absorption 0.2 %, temp. resistance 62 °C

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