Erkoflex-82 Sticks

Erkoflex-95 Sticks

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Erkoflex-82/-95 Sticks, transparent

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EN: Instructions for use

Erkoflex-82/-95 Sticks, transparent

The general purpose of Erkoflex-82/-95 Sticks is the manufacture of appliances for dental and orthodontic treatment.

Application (material)	Intended use	Indication	Patient groups
Occlusal splints (Erkoflex-95 stick in combination with Erkoflex 95) - MD	CMD treatment	CMD	Adults, teenagers, children
Positioner (Erkoflex-82 stick in combination with Erkoflex) - MD	Positioning of teeth	Misaligned teeth	Adults, teenagers

Contraindications: If an allergic reaction to plastics is already known, a specific allergy to Erkoflex-82/-95 sticks (EVA) should be ruled out.

Attention: Observe storage symbols.

User: Dental professionals

Intended use: Erkoflex-82/-95 sticks are recommended as a medical device exclusively for the above-mentioned purposes.

Note: The use is subject to responsibility and is based on the recommendation/prescription of the dentist. Allergic reactions are unlikely, but cannot be ruled out.

Combination products: Erkoflex for Erkoflex-82 sticks, Erkoflex-95 for Erkoflex-95 sticks. Special attachment for hot glue gun 177 010, commercially available hot glue gun (power ≥ 500 W).

Cleaning: Dental splints can be cleaned with Oxydens cleaning tablets. Also with curd soap, water and a toothbrush. Toothpaste is not suitable. Do not heat above 50 °C.

Please rinse the splint with clear water before inserting it. As the user, please inform the patient about the cleaning options.

Further information: Thermoforming technology brochure and videos, www.erkodent.com

Serious incidents in connection with Erkoflex-82/-95 sticks must be reported. The user must inform the competent authority and the manufacturer.

Performance characteristics (Circa values)

Erkoflex-95 sticks: Hardness 95 Shore A, density 0.93 g/cm³, modulus of elasticity 70 MPa, elongation at break 450 %, water absorption 0.2 %, temperature resistance 62 °C

Erkoflex-82 sticks: Hardness 82 Shore A, density 0.95 g/cm³, modulus of elasticity 20 MPa, elongation at break 850 %, water absorption 0.5 %, temperature resistance 54 °C

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