Kneton Lab 95

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Kneton Lab 95

Instruction for use

Laboratory moulding material on polysiloxane based, condensation cured, very high consistency, mouldable Kneadable silicone putty to form sectional moulds for dental repairs in the laboratory, suitable when using cold and hot curing acrylic resins, etc.

See enclosed documents

Read instructions for use

Keep away from sunlight

Keep dry

5 °C/41 °F — 35 °C/95 °F

Recommended storage temperature

15 °C/59 °F — 25 °C/77 °F

Recommended working temperature

Dosage:

To 1 level measuring spoon of **Kneton Lab 95** (about 17 g) 2.0 cm of hardener paste (0.5 g) are added.

Application:

Mix **Kneton Lab 95** with the hardener paste for 30 - 45 sec.

Hint:

Higher room temperatures will accelerate the setting.

Technical Data:

- mixing can be effected manually or by machine
- mixing time: **30 – 45 sec.**
- total working time: **1:30 min.**
- setting time: **3 min.**
- Shore-A hardness: **approx. 95**

Attention:

Avoid eye contact with the hardener;
avoid stains on clothes.

The user of our products is the only person responsible for the use and the application of our products. Our verbal and written advice has to be considered as advice without obligation also regarding possible protection rights of third parties. The user is not released from the obligation to check our products personally concerning suitability for the intended application. The perfect quality of our products is guaranteed according to our General Terms and Conditions.

In any case and for all damage our liability, is limited to the value of the goods supplied by us.

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Erkodent Erich Kopp GmbH • Siemensstraße 3 • 72285 Pfalzgrafenweiler, Germany