Silensor-sl

Bauteile • parts • composants
componenti • componentes • komponenty

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EN: Instructions for use

The general purpose of the Silensor-sl components is the manufacture of the Silensor-sl anti-snoring splint, an appliance for dental treatment.

Application (material)	Intended use	Indication	Patient groups
Silensor-sl parts - MD	Therapy of OSA and ronchopathy	OSA and ronchopathy	Adults, teenagers

Contraindications: If an allergic reaction to plastics is already known, a specific allergy to the Silensor-sl components (polyamide) should be ruled out.

Attention: Observe storage symbols.

User: Dental professionals.

Intended use: Silensor-sl components are recommended as a medical device exclusively for the above-mentioned intended use.

Note: The use is subject to responsibility and is based on the recommendation/prescription of the dentist. Allergic reactions are unlikely, but cannot be ruled out.

Incorrect processing can lead to fractures. There is a risk of aspiration in the case of small fractures.

Combination products: Erkodur and/or Erkoloc-pro.

Cleaning: The Silensor-sl can be cleaned with Oxydens cleaning tablets. Also with curd soap, water and a toothbrush. Toothpaste is not suitable. Do not heat above 50 °C.

Please rinse the splint with clear water before inserting it. As the user, please inform the patient about the cleaning options.

Further information: Thermoforming technique brochure and videos, www.erkodent.com

Serious incidents in connection with Silensor-sl components must be reported. The user must inform the competent authority and the manufacturer.

Performance characteristics (Circa values): Density 1.0 g/cm³, modulus of elasticity 1600 MPa, elongation at break > 50 %, water absorption 3.0 %, temp. resistance 115 °C

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