

exMod-pro

INFO
EN

Device for easy separation of
thermoformed aligner foils from print models



exMod-pro
order no. 173 200



 Erkodent Erich Kopp GmbH
Siemensstraße 3 • 72285 Pfalzgrafenweiler • Germany
info@erkodent.com • www.erkodent.com



exMod-pro
with mounted
print model and
thermoformed
foil, ready for
separation.

Suitable for all
aligner foils.



Print model with
hook-on pockets




Clamping template for foil
thicknesses ≤ 0.6 mm and for
particularly soft foils.


exMod-pro-info-EN-08-04-26



Hang the thermoformed
foil and print model with
the hook-on pockets onto
the hook-on pins and
press down gently.

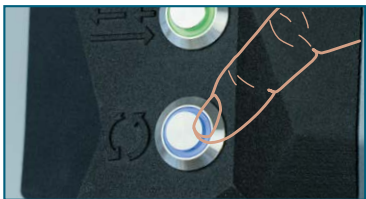

Press the blue illuminated
switch to initiate the
separation process.
This process runs
automatically.

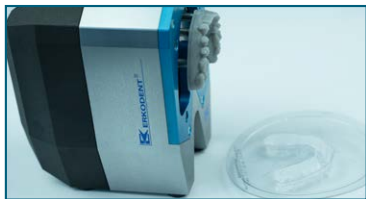

The foil is separated from
the model and the model
can be removed.