

Helmet strap for American Football



PLAY SAFE®


+ **PLAY SAFE® triple**

Instructions:

- ▶ **1.** Degrease the end of the helmet strap and the spot at the mouthguard where the strap shall be fixed (degreasing agent order no. 613 050).
- 2.** Heat the spot at the mouthguard where the strap shall be fixed with the hot-air burner (order no. 177 540).
- 3.** Heat the end of the helmet strap with the burner and ...
- 4.** ... press it on the mouthguard immediately. Press bulges on flatly with the aid of a smooth, rounded instrument.

ATTENTION! These works must always be executed on the model, otherwise you risk to deform the mouthguard.

584 025 deep black, 10 pieces

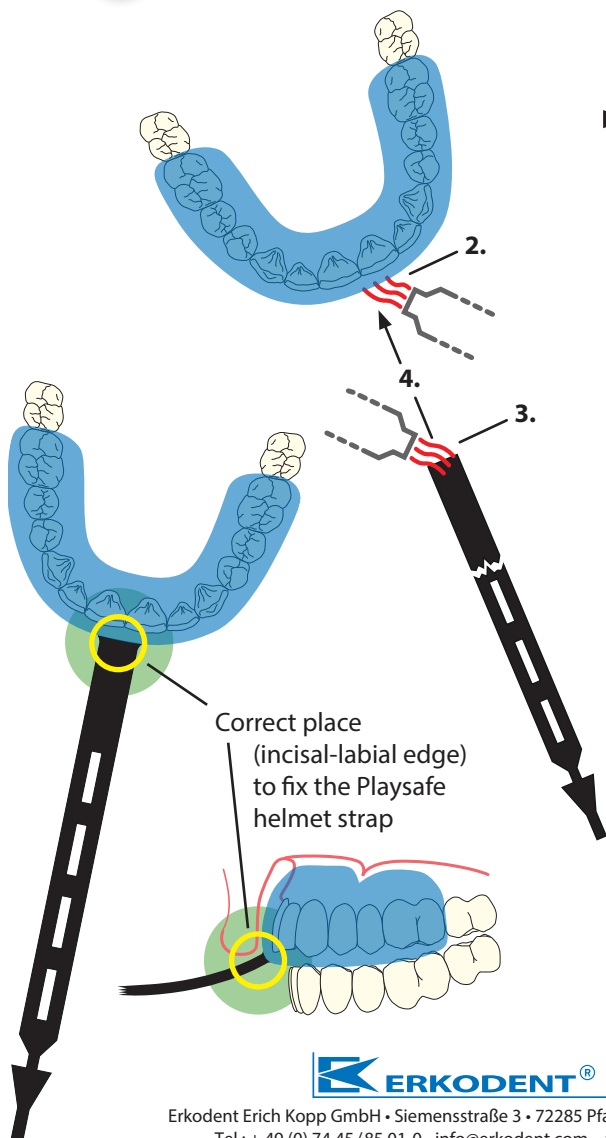

ERKODENT®

Erkodent Erich Kopp GmbH • Siemensstraße 3 • 72285 Pfalzgrafenweiler • Germany
Tel.: + 49 (0) 74 45 / 85 01-0 • info@erkodent.com • www.erkodent.com